

BMC F2

FULL FACE BiPAP MASK

Comfort. Performance. Better Sleep.

The BMC F2 Full Face BiPAP Mask is designed for patients who breathe through their mouth or nose during sleep therapy. It ensures a secure seal, maximum comfort and effective therapy throughout the night.



SUPERIOR SEAL
LEAK-FREE PERFORMANCE



ULTRA COMFORT
SOFT & LIGHTWEIGHT DESIGN



EASY TO USE
SIMPLE FIT & ADJUSTMENT



DURABLE & SAFE
PREMIUM QUALITY MATERIALS



COMPATIBLE WITH

BMC
BiPAP & CPAP
SYSTEMS

KEY FEATURES



Ergonomic Full Face Design
Covers both nose and mouth for effective therapy even if you breathe through your mouth.



Soft Silicone Cushion
Medical grade silicone ensures a comfortable, gentle seal without pressure marks.



Stable & Secure Fit
Adjustable headgear with strong clips provides a secure fit and minimizes leaks.



Quick Release Elbow
360° swivel elbow allows flexible movement and easy tubing connection.



Clear & Lightweight Frame
Offers a clear field of vision and reduces facial contact for enhanced comfort.



Easy to Clean & Maintain
Simple to assemble, disassemble and clean for daily hygiene.

SPECIFICATIONS

Product Name	BMC F2 Full Face BiPAP Mask
Model	F2
Mask Type	Full Face Mask
Material	Medical Grade Silicone & Polycarbonate
Cushion	Soft Silicone Cushion
Headgear	Adjustable, Elastic Headgear
Elbow	360° Rotating Elbow with Quick Release
Compatibility	Compatible with BMC BiPAP & CPAP Machines
Size	Standard (One Size Fits Most)
Color	Transparent & Grey
Weight	Lightweight Design
Usage	Single Patient Use
Cleaning	Wash with mild soap & water, air dry

AVAILABLE BENEFITS

- ✓ Ensures effective therapy with minimal air leaks.
- ✓ Gentle on skin for long-term comfort.
- ✓ Promotes better sleep and improved therapy compliance.
- ✓ Suitable for daily use at home or clinical settings.



HOW TO USE

1. Attach the headgear to the mask frame.
2. Place the mask over your nose and mouth.
3. Pull the headgear over your head and position the straps evenly.
4. Adjust the straps for a comfortable, leak-free fit.
5. Connect the tubing to the swivel elbow and your BiPAP machine.

SAFETY & CARE

- For single patient use only.
- Clean the mask daily with mild soap and lukewarm water.
- Do not use strong detergents or solvents.
- Air dry away from direct sunlight.
- Replace the cushion and parts regularly for optimal performance.

IDEAL FOR PATIENTS

- Who breathe through their mouth or nose
- Requiring BiPAP therapy for conditions like OSA, COPD & Respiratory Failure
- Seeking comfort and reliable performance throughout the night