



Deckmount Vented BiPAP Nasal Mask



COMFORTABLE NASAL INTERFACE FOR EFFECTIVE CPAP / BiPAP THERAPY

The Deckmount Vented BiPAP Nasal Mask is designed to deliver comfortable, non-invasive ventilation support. It features a secure nasal fit and built-in venting for stable airflow, ensuring effective therapy performance. Ideal for home care, sleep therapy, and clinical use.



1 PRODUCT FEATURES

- Vented nasal mask design
- Soft and comfortable nasal cushion
- Lightweight transparent frame
- Adjustable elastic headgear
- Secure fit for improved therapy support
- Easy connection to BiPAP / CPAP tubing
- Clear, user-friendly design
- Suitable for daily use

2 SPECIFICATIONS

Product Name	Deckmount Vented BiPAP Nasal Mask
Type	Vented Nasal BiPAP Mask
Patient Type	Adult
Coverage	Nose Only
Cushion Material	Soft Medical Grade Silicone
Frame	Lightweight Transparent Frame
Headgear	Adjustable Elastic Headgear
Vent Type	Built-in Vented Design
Color	Transparent with Grey Headgear
Compatibility	BiPAP / CPAP Therapy Devices
Use	Reusable with proper cleaning



FOR PRODUCT
QUERIES
CALL US NOW

9355100499

3 BENEFITS

- Supports stable airflow for effective therapy
- Comfortable for longer therapy sessions
- Helps reduce pressure discomfort
- Easy to wear and remove
- Ideal for home care and hospital use

4 HOW TO USE

- Connect the mask to the CPAP / BiPAP tubing
- Position the mask over the nose
- Pull the headgear over the head
- Adjust the straps for a comfortable fit
- Start therapy and check the seal



5 CARE & CLEANING

- Clean the cushion regularly with mild soap and water
- Keep away from direct heat and sunlight
- Inspect connectors and straps regularly
- Replace worn or damaged parts when needed

6 BEST FOR

- Home Care
- Hospitals
- Clinics
- Sleep Therapy

7 WHY CHOOSE US?

- Trusted Care
- Quality You Can Rely On
- Care That Comes Home

